## RETENTION GUARD™

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**RETENTION GUARD™**  
(Retention Suture Protective Plate)

**INSTRUCTIONS FOR USE**

**CE 0086**

|                |           |
|----------------|-----------|
| <b>STERILE</b> | <b>EO</b> |
|----------------|-----------|

RM/N 4080  
Rev. 052518

**Description**

The Retention Guard™ is a foam padded protective plate with holes for placement of retention sutures. Retention Guards are 3 inches long and are packaged two (2) per sterile pouch. Ten (10) sterile pouches per box. Contents are sterile unless the package has been opened or damaged.

**Intended Use**

Retention Guards may be used for temporary, external anchoring of retention sutures in position, and for prevention of suture pull through.

**Contraindications**

None known

**Warnings**

FOR EXTERNAL USE ONLY

FOR SINGLE PATIENT USE ONLY

Federal Law restricts sale of this device to, by, or on the order of a physician. Use of this device should not otherwise affect the standard of care for the condition being treated.

**Precautions**

Excessive pressure by suture attachments between the Retention Guards may cause skin necrosis. Prolonged use may cause skin maceration. If additional skin protection is desired the surgeon may choose to apply a hydrocolloid dressing beneath the Retention Guard.

**RETENTION GUARD™**  
(Retention Suture Protective Plate)


**CE 0086**

**STERILE EO**

RM/N 4080  
Rev. 052518

**Instructions for Use**

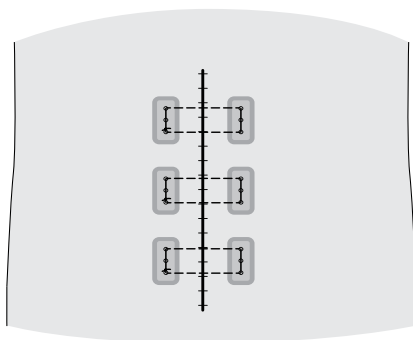Retention Guards are typically positioned on opposite sides of the wound as shown. Retention sutures are applied per surgeon technique. After use Retention Guards are removed and discarded as biohazardous waste.

3 inch Retention Guards used for securing retention sutures in position.

Manufacturer:  
Preservation Solutions, Inc.  
1099 Proctor Drive  
Elkhorn, Wisconsin  
53121 USA  
262-723-6715 tel.  
262-723-4013 fax

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31(0)70.345.8570 tel.  
+31(0)70.346.7299 fax

Sales Contact:  
Starsurgical, Inc.  
7781 Lakeview Drive  
Burlington, Wisconsin  
53105 USA  
262-539-3564 tel.  
262-539-2096 fax  
[www.starsurgical.com](http://www.starsurgical.com)

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