STARGUARD™ (Suture Retention Plates)

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Instructions for Use

Description

The StarGuard™ is a foam padded protective plate with evenly spaced suture holes. StarGuards are supplied in 7 and 9 inch lengths and are packaged according to size: the 7 and 9 inch lengths are packaged four (4) per sterile pouch. Contents are sterile unless the package has been opened or damaged.

Intended Use

StarGuard protective plates may be used for temporary, external anchoring of sutures in position, and for prevention of suture pull through.

Contraindications

None known

Warning

FOR EXTERNAL USE ONLY

FOR SINGLE PATIENT USE ONLY

Federal Law restricts sale of this device to, by, or on the order of a physician. Use of this device should not otherwise affect the standard of care for the condition being treated.

Precautions

Excessive pressure applied to the StarGuard by suture attachments may cause skin necrosis. Prolonged use may cause skin maceration. If additional skin protection is desired the surgeon may choose to apply a hydrocolloid dressing beneath the StarGuard.

Instructions for Use

StarGuards are typically positioned on opposite sides of the wound. Sutures are applied per surgeon technique. Figure 1 illustrate possible uses. StarGuards may be trimmed by peeling back the protective foam padding and cutting the plastic, allowing for the foam to extend beyond the trimmed plastic edge. StarGuards are removed and discarded as biohazardous waste when sutures are removed.

STARGUARD™
(Suture Retention Plates)

CE 2797

STERILE EO

RM/N 4106

Rev. 051120

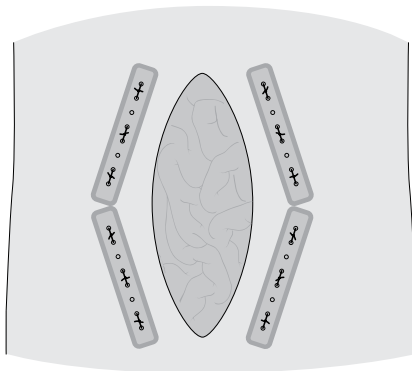
Figure 1 7 or 9 inch StarGuards used to anchor sutures during open wound closure.

Manufacturer:
Preservation Solutions, Inc.
1099 Proctor Drive
Elkhorn, Wisconsin
53121 USA
262-723-6715 tel.
262-723-4013 fax

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31(0)70.345.8570 tel.
+31(0)70.346.7299 fax

Sales Contact:
Starsurgical, Inc.
7781 Lakeview Drive
Burlington, Wisconsin
53105 USA
262-539-3564 tel.
262-539-2096 fax
www.starsurgical.com

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